

# How to Operate the MiniTracker or the Multi-Scan 125 Scanner



## To Operate the Scanner:

**The antenna of the scanner is located in the tapered end. Marked 'READ HERE'. This is the end of the scanner that 'reads' the chip number.**

1. Slide the On/Off button to ON. You will hear two short 'beeps.' The scanner will display 'AVID ID READY'.
  2. Press and lightly hold down the small round button, called the Read button. The Scanner will display 'LOOKING'.
  3. Keep applying gentle pressure on the Read button while you pass the scanner over the injection site between the shoulders...in a circular motion. Use a close sweeping motion about two inches from the surface of the animal.
  4. When a chip is detected, you will hear two 'beeps' and the scanner will display the chip number in the viewing window.
  5. You may release the read button. The microchip number will remain displayed in the window until you scan again or turn off the scanner.
- Note: If you release the Read button prior to locating a chip, the scanner will display NO ID FOUND. Simply press the Read button again.
  - Note: An alarm will sound after three minutes if the scanner is left on.

**Note:** The regular MiniTracker reads only the AVID nine-digit SecureChip. (125Khz)  
The MultiScan 125 MiniTracker reads any brand of 125 kHz chip.

125 kHz brand names you may recognize are AVID® and HomeAgain™, Destron, Digital Angel, and 24 Pet Watch.

Your scanner has a full One Year Warranty.

Your scanner operates from the power of one 9-volt battery.  
Please replace with a **high quality alkaline**. Your scanner has a built in low battery sensor.  
If your scanner displays LOW BATTERY, you have a few days left before it wears down completely



**Microchip ID Systems, Inc.    [www.microchipIDsystems.com](http://www.microchipIDsystems.com)  
Call with any problems.... (800) 434-2843**